Flairesse

Prophylaxis paste

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English

Product description

Tooth cleaning and polishing paste for professional tooth cleaning and polishing of the tooth surface as part of prophylaxis treatment. Flairesse Prophylaxis paste comes in a Unit Dose or a tube and is available in various flavors. The pastes' three different abrasion grades enable you to respond to each patient's situation, depending on the level of cleaning required.

Indications

- Removal of superficial soft and hard plaque.
- Removal of extrinsic stains (e.g. stains from coffee, tea or tobacco).

Contra-indications

Do not use the material

- in cases of allergies to any of the ingredients or in cases of contact allergies,
- if the patient's control over their swallowing reflex is not guaranteed.

Basic safety instructions

- For dental use only!
- Keep out of reach of children!
- Do not swallow!
- Avoid contact with the eyes! In the event of inadvertent contact with the eyes, immediately rinse eyes thoroughly with plenty of water and consult a physician if necessary.

Side effects

Swallowing excessive amounts of the material may cause nausea.

Application system

- Unit Dose
- Tube

Instructions for use

- All disposable prophylaxis contra-angles or polishing cups available on the market may be used.
- DMG recommends using the prophylaxis pastes one after the other (decreasing the abrasion grade). Use brushes for occlusal surfaces.
- The Unit Dose and the tube are fitted with a tamper-proof seal. Make sure that the seal is intact before first use.

Recommended use

Caution! Incorrect use may result in overheating.

- Ensure that there is sufficient material in the polishing cup used.

Caution! Risk of swallowing if saliva suction insufficient.

- Saliva must be suctioned away thoroughly during treatment.

Using the paste from the Unit Dose

Please note: Each Unit Dose holds enough material for one prophylaxis treatment. Each Unit Dose may only be used for one individual patient.

- Peel off the tamper-proof seal from the Unit Dose using the tag and discard.
- Insert the Unit Dose into the appropriate prophylaxis holder.
- To fill, set the polishing cup to low speed, hold it in the prophylaxis paste and pick up the material.
- Use the polishing cup to work in the prophylaxis paste for 20s per tooth at 3000 to 5000rpm.
- Suction away excess prophylaxis paste and get the patient to spit it out.

Applying the paste from the tube.

Please note: To prevent the material from drying out, close the tube as soon as you have expressed the material.

- When using a tube for the first time, unscrew the cap, peel off the tamper-proof seal using the tag and discard.
- Express a sufficient quantity of the material onto a mixing block.
- To fill, set the polishing cup to low speed, hold it in the prophylaxis paste and pick up the material.
- Use the polishing cup to work in the prophylaxis paste for 20s per tooth at 3000 to 5000rpm.
- Suction away excess prophylaxis paste and get the patient to spit it out.

Ingredients

Pumice, glycerine, water, o-phosphoric acid (< 2 %), flavors and scents, additives, sodium fluoride (2.72 %, corresponds to 1.23 % fluoride ions/12.300 ppm).

Free from aspartame, gluten, saccharine.

Contains Xylitol.

Storage and shelf life

- Store at room temperature (15 to 25 °C/59 to 77 °F)!
- Do not store in the refrigerator!
- Protect against exposure to direct sunlight!
- Do not use after the expiration date!

Available product sizes

Unit Dose

One pack contains 200 individual portions @ 1.8 g. Available in the following flavors and grit sizes:

Mint	
Fine grit	REF 220414
Medium grit	REF 220415
Coarse grit	REF 220416

Tube

One pack contains 1 tube @ 75 ml. Available in the following flavors and grit sizes:

Mint	
Fine grit	REF 220417
Medium grit	REF 220418
Coarse grit	REF 220419

Melon	
Fine grit	REF 220423
Medium grit	REF 220424
Coarse grit	REF 220425

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